



Quick Des N/Quick Des N Lemon

Alcoholic rapid disinfectant

- safe disinfection for daily routine
- comprehensive, rapid effectiveness
- very good material compatibility

Read label before use.

Type of preparation

Quick DES N Lemon is a ready-to-use, alcoholic rapid - disinfectant based on ethanol for material-friendly disinfection and cleaning of medical devices and medical equipment as well as surfaces of all types. The product is particularly ideal for areas near patients, where a universal spectrum of activity is required. In addition to effective and rapid disinfection, the product has very good material compatibility and leaves no residues.

Composition

100 g solution contains: 45 g ethanol

Chemical-physical data

Appearance of the concentrate clear, colourless solution
 pH-value: 5,5 - 6,5
 Flashpoint: 24°C

Application areas

According to the Biocidal Products Regulation (BPR):
 Rapid disinfection and cleaning of medical equipment and surfaces of all types, mainly in near patient areas.

According to EU-Medical Devices Regulation:
 Rapid disinfection and cleaning of non-invasive medical equipment, especially for sensitive areas of medical devices. □

Application

For complete wetting use the undiluted solution. Keep the surfaces moist for the duration of the required contact time. According to EU-Medical Devices Regulation: Users/patients are obligated to report any serious incident that has occurred in relation to the product to the manufacturer and the responsible authority of the EU member state in which the user/patient is established.

Application note

Quick DES N Lemon can be used as a wipe disinfection or spray application.



VAH listed

Exposure times

Recommended use for surface disinfection:

bactericidal, levorocide:	2 mins
tuberculocide:	30 s
mycobactericide:	1 mins
limited virucidal:	1 mins
limited virucidal PLUS:	3 mins

Delivery forms

Single pack	Sales unit	Ref.-No.
Duftneutral		
1 Liter bottle	1 x 1 L	09920-N
5 Liter bottle	1 x 5 L	09921-N
10 Liter canister	1 x 10 L	09922-N
Lemon		
10 Liter canister	1 x 10 L	09925-N

Quick Des N: baua: Reg.-No N-97772
Quick Des N Lemon: baua: Reg.-No N-97774

Expertise

Our products have been fully tested. Upon request we will be pleased to provide you with our report folder.

Classification acc. to Regulation (EC) Nr. 1272/2008



Precautionary and hazard statements

Flammable liquid and vapour. If medical advice is needed, have product container or label at hand. Keep out of reach of children.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour/spray.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only by personnel with corresponding specialist knowledge according to national directives. Use disinfectants safely.

Always read label and product information before use.



AMPri Handelsgesellschaft mbH
 Benzstr. 16 • 21423 Winsen (Luhe) • Germany
 Tel. +49 (0)4171- 8480-0 • www.ampri.de • info@ampri.de



bioSight GmbH
 Brasselsbergstr. 3 • 34132 Kassel • Tel.: +49 (0)561 81678111

Stand: 11-2023